


Roller: Next / Prior Image
 Ctrl + Roller: Variable Image Mag
 Both Mouse Buttons: Pan Image

Press & Hold: Page Thru Images
 Double Click: Select Images for Summary Series

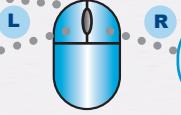

Press & Hold: Changes Level
 Changes Window
 Double Click: Dropdown Menu

Normal Mode F1	Trackball Mode F2	Measure Distance F3	ROI F4
-------------------	----------------------	------------------------	-----------

User Annotation F5	Select Image Sum Series F6	Process Summary Series F7	Magnifying Glass Toggle F8
-----------------------	-------------------------------	------------------------------	-------------------------------

Image Intersect Display F9	3D Mode (Optional) F10
-------------------------------	---------------------------

Keyboard Operation

← ↓ →	Advance One Image
↑	Advance One Row
+ -	Next / Prior Image Series
Home	Beginning of Series
End	End of Series
Page Up	Advance One Screen
Page Dn	Back One Screen
0 - 9	Mouse Sensitivity Adjust.
L M	Less / More Annotation